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EXTOL® CRAFT 9723

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Translation of original user's manual

EXTOL® CRAFT 9723

Formed Disposable Hygienic Face Mask

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www.extol.cz

PROCEDURE FOR PUTTING ON (FIG. 1-3):

1. Place the face mask under your chin so that the metal strip points towards your eyes.
2. Place the face mask against the face with the metal strip over the nose and while holding the face mask in place use your other hand to pull the rubber band over your head.
3. Gently shape the metal strip so that the face mask sits flush on the face.
4. Check the tightness around the contact line of the face mask by breathing in deeply. In the event of a leak, adjust the shape of the metal strip or the mask's position on the face.

⚠ ATTENTION

- For the proper function of the face mask, it is important that there are no whiskers or hair on the contact line.

In the event of an allergic reaction to the face mask material or breathing difficulties resulting from the increased breathing resistance, do not use the face mask.

INTENDED USE:

- The face mask is intended only for increasing the hygienic standard, e.g. when handling objects, e.g. when the user has symptoms of a cold or hay fever when harvesting fruit in the garden, etc.. The face mask does not however, prevent the spreading of infection from the user and does not protect the user against air-borne infections. Thus, this face mask is not a substitute for a medical face mask!

⚠ ATTENTION

- The face mask is not intended to provide protection to the respiratory tract of the user against gases, fumes and solid or liquid aerosols and thus is not a substitute for respirators FFP1, FFP2 and FFP3! The product does not protect the user against harmful substances and as such must not be used as a personal protective aid!
- The product is not defined pursuant to Act. No. 22/97 Coll. and is not subject to Council Directive 89/686 EHS stipulating the requirements for personal protective aids.
- Short term use of the face mask is assumed as is adherence to basic rules of hygiene for use, where it is assumed that each user will have their own face mask. The face mask is not intended to be washed and disinfected.

STORAGE

- Store the product in a dry place at a temperature of -5°C to 40 °C. Protect it against moisture and direct sunlight.